


N'GO — *line*

Instruction Manual

N'GO301 - N'GO303 - N'GO304F - N'GO304 - N'GO308

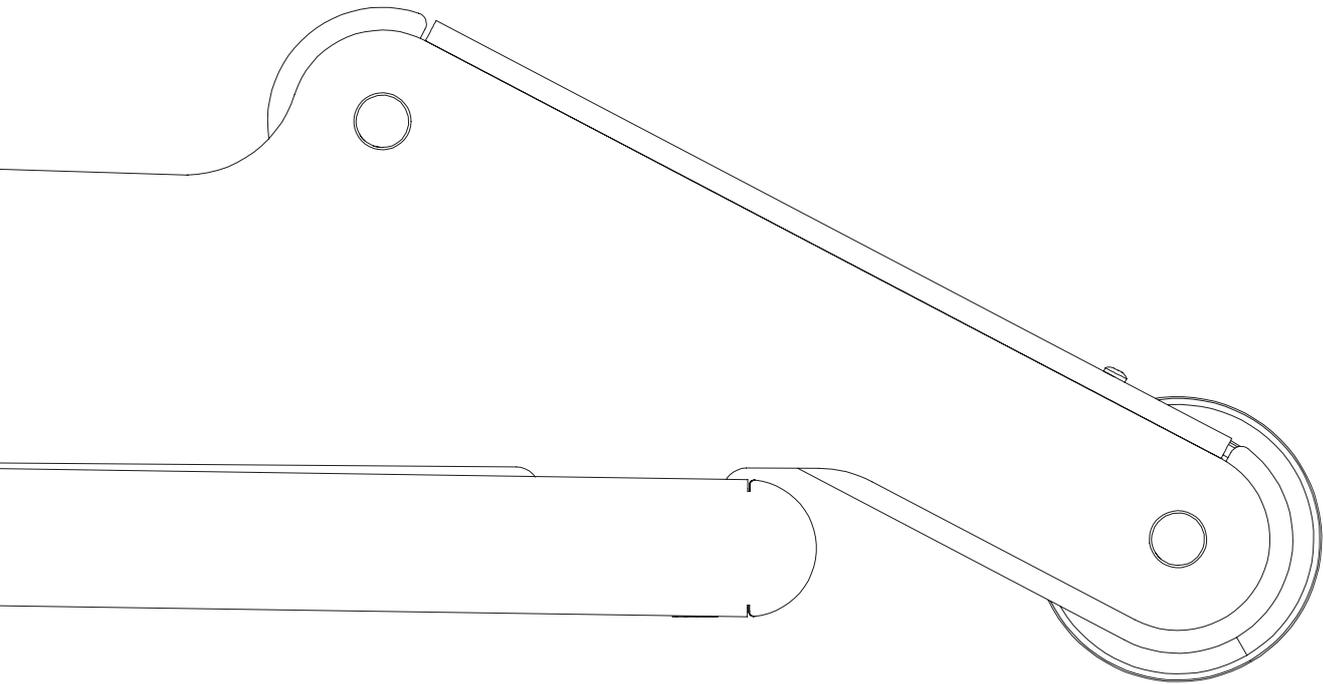

Instruction Manual

Naggura N'GO301 - N'GO303 - N'GO304F - N'GO304 - N'GO308

Table of contents

-  1. Introduction
-  2. Use of the manual
-  3. Warranty
-  4. Liability
-  5. EC Declaration of conformity
-  6. Warnings and general instructions
-  7. Dimensions
-  8. Settings
 - Wheels
 - Gas piston levers
 - Perimeter foot switch
 - Handlebar support
 - Paper roll holder
 - Breather hole
 - Armrests
 - Levelers
-  9. Maintenance
-  10. Troubleshooting



Introduction

CONGRATULATIONS! You took NAGGURA home!

Thank you for having acquired a NAGGURA product and for the trust shown in us.

We are sure that this treatment table will turn into the pillar of your work and will more than fulfil your most demanding work requirements. NAGGURA's experts are in the lead of a project created with great devotion. Every single detail has been cared to join technology, performance and design altogether in a single table. Our aim is designing and manufacturing INTUITIVE, DYNAMIC and HI-TECH tables of a DARING and UNIQUE design.

Thanks for your choice and start enjoying the NAGGURA experience.

TM naggura



Use of the manual

Please make sure that this instructions manual is always accessible to the personnel involved with the use of this equipment.

The manual describes the correct use of the different models and types.

Some texts and/or illustrations are not relevant to your Naggura couch since it depends on your product choice at the purchase time.



Warranty

Weelko Barcelona, S.L. guarantees the material against manufacturing defects for 2 years from the delivery of the material. The invoice issued by Weelko Barcelona S.L. will become the warranty certificate so it needs to be kept.



Liability

The manufacturer cannot be held responsible for any consequences incurred by the user, the patient or by the couch itself as a result of e.g.

- a. Misuses of the table.
- b. Damages caused by a mishandling when unpacking the table, a negligence, a fortuitous event or natural wear of the product.
- c. Handling or repairing without the Naggura technical service's knowledge.
- d. Upholstering carried by the customer.
- e. Claims without invoice.
- f. Power surges or inappropriate voltage connections.
- g. Upholsteries stained or damaged caused by chemical or acid products.

After a period of 10 (ten) years after introduction of the couch (or any of its accessories), the manufacturer can no longer be held responsible for any flaws in the couch or its accessories and he will no longer be obliged to supply any spare parts or accessories.



EC Declaration of conformity

Naggura products are produced and marked in conformity with the directives hereby stated :

Low voltage : 2014/35/CE
Electromagnetic compatibility : 2014/30/CE




These products are manufactured according the essential requirements established in the European Medical Devices Directive 93/42/EEC.

The N'GO line has a guarantee of 2 years. The manufacturer will not be held responsible for the results of maintenance or repairs performed by unauthorised persons. For further information on guarantee terms, please contact your distributor.




Warnings and general instructions

1. When you receive the order, check first whether the product has been damaged during transport. Report your dealer immediately about any damage, defect or non-conformity with the order received.
2. Keep the packaging and store it during the first 20 days.
3. Read this manual carefully before using the equipment.
4. The maximum loading weight is 200 kg.
5. Do not place the hands in the moving parts of the treatment table when performing any electrical adjustment.
6. Guide the backrest section with the other hand when adjusting the backrest by gas levers.
7. Make sure that the unit is correctly plugged to an earth intake following the local and national electrical requirements and standards.
8. Although the treatment couch is equipped with wheels, it cannot be used for the transport of patients.
9. Keep the table out of high-humidity environments.
10. If the table gets wet accidentally, unplug the table immediately.
11. Naggura treatment tables are only meant to be used by professional personnel in physiotherapy, rehabilitation and massage fields. Avoid any use of the equipment by unauthorised non-skilled personnel.
12. Place the table in a space with free access form the 4 sides. Do not place the table against the wall or against any obstacles.
13. Do not leave children unattended alone near the table.
14. Change the bed cover or protecting paper after each patient in order to guarantee the hygiene of the treatment.
15. Whenever it is necessary to lift the bed, DO NOT take it by the upholstered section. You must take it by the structure, as it is shown in the pictures below.

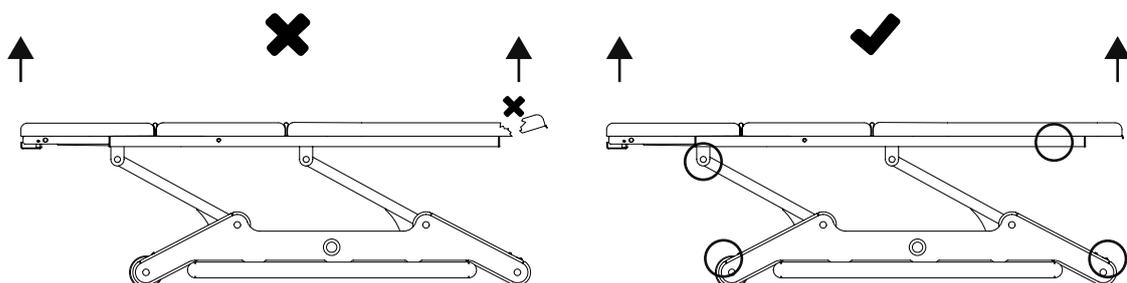






Product dimensions in movement (cm)

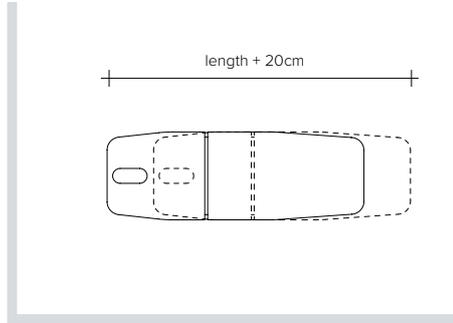


Power cord length (cm)

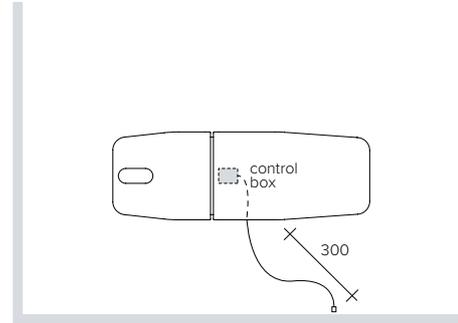

Pack dimensions (cm)

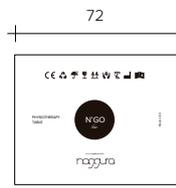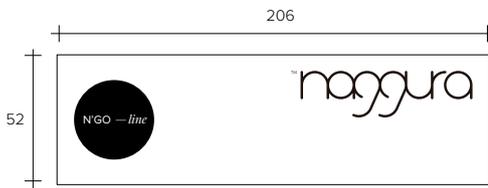

Handling and transport

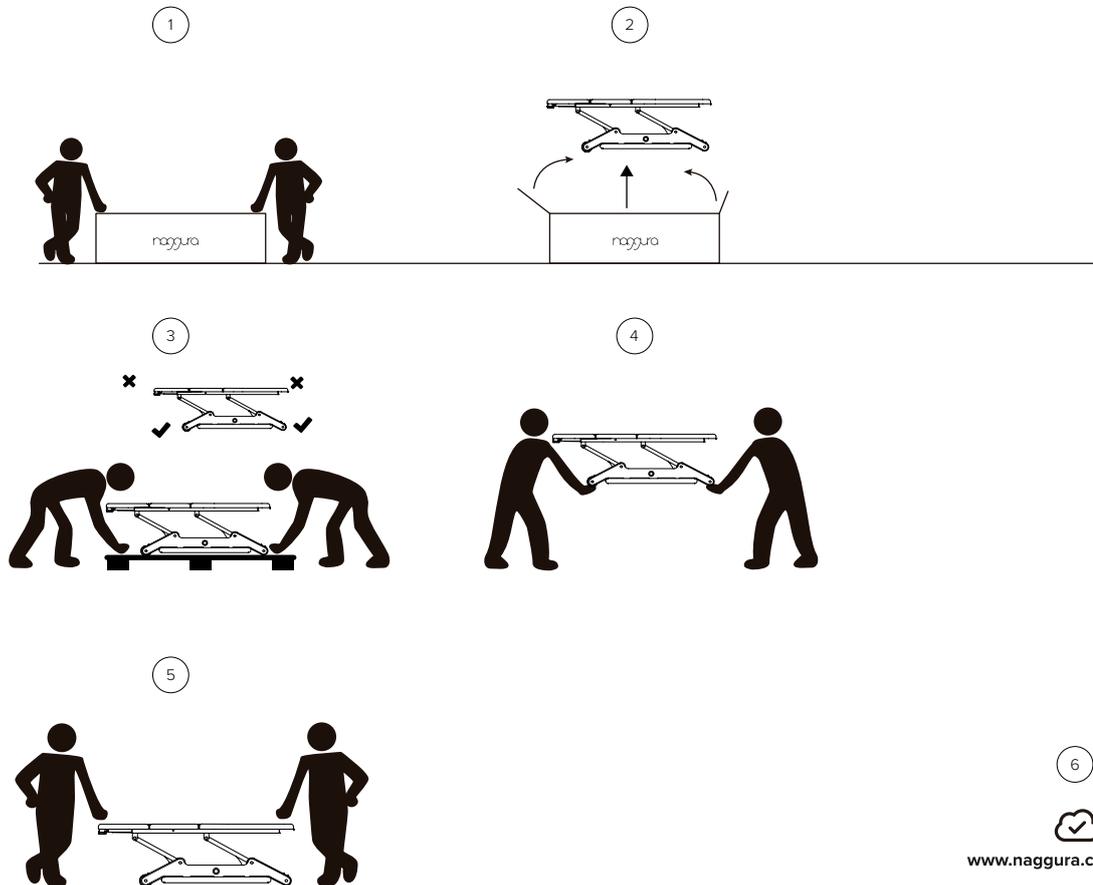









Essential Settings

1. Wheels
2. Gas piston levers
3. Perimeter foot switch
4. Handlebar support
5. Paper roll holder
6. Breather hole
7. Armrests
8. Levelers
9. Safety Lock

☞

☞ Optional


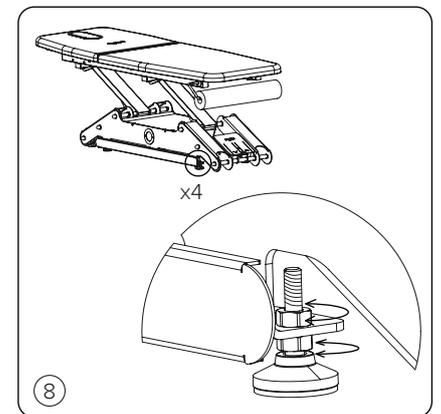
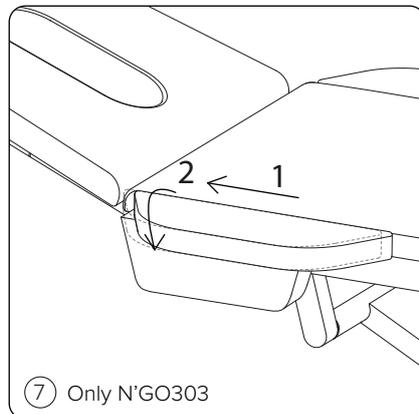
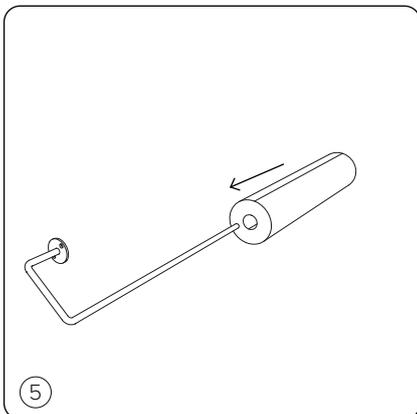
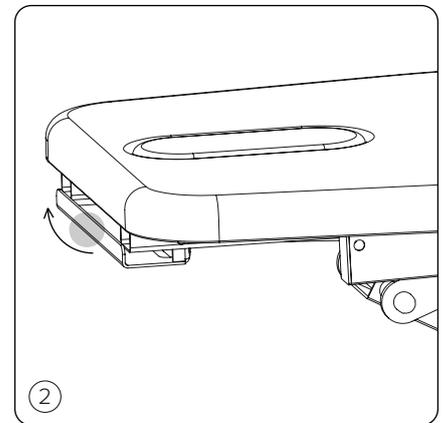
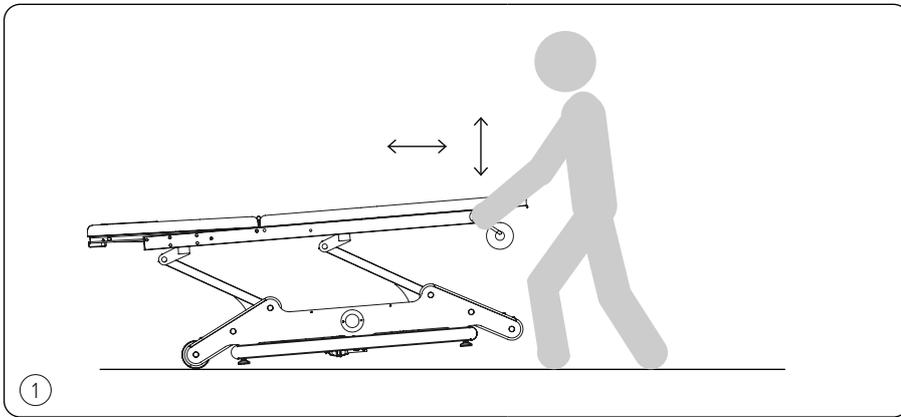
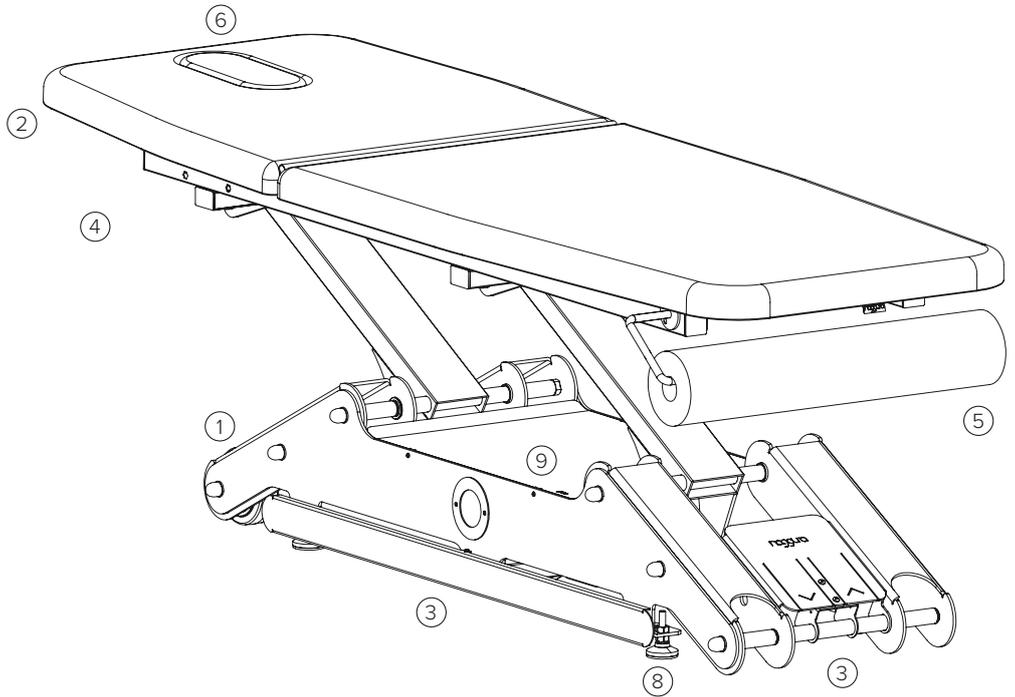




Essential Settings

Perimeter foot switch ③

The height of the table can be adjusted from the 4 sides of the table thanks to a very intuitive perimeter device. The mechanism on both sides consists of 2 grey long overhanging bars which are fully integrated in the base making the design simply beautiful. When the physiotherapist presses the end of the bar with his foot, the table goes up or down; the opposite end of this bar is used to activate the opposite movement (up or down). Do not press just in the middle of the bar, since this is the supporting axis of the swinging bar and, at this point, the height the mechanism does not work. In the front and rear part of the table, the height motion is operated by means of 2 foot switches for both up and down movements respectively. The arrows show the sense of the movement (up or down). In the case of N'G0304-2 (2 engines), the perimeter foot switch controls the height while the second movement is controlled by 2 buttons placed one side of the table.

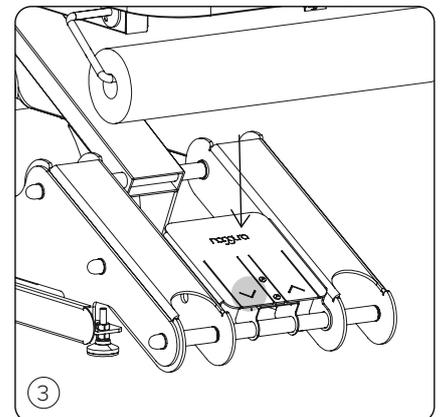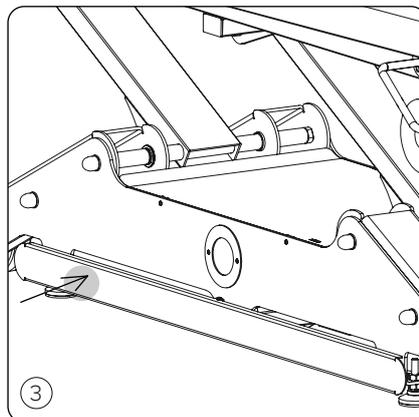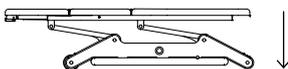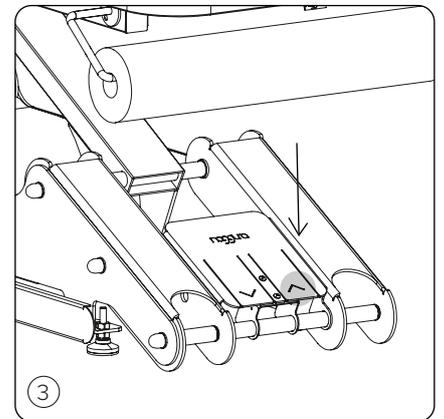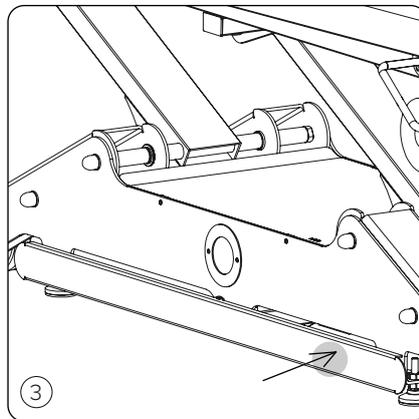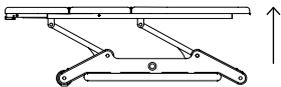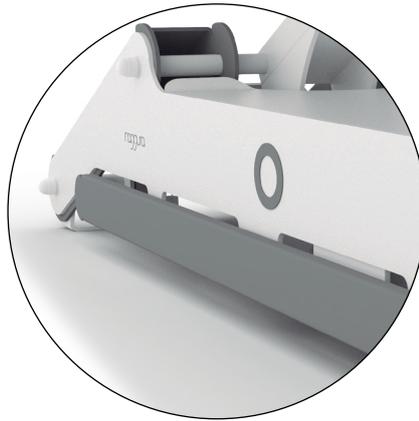




Essential Settings

Power input

Voltage and frequency input shall be AC 100-120, 50/60Hz or 220-240V, 50/60Hz. Plug the bed on the power supply properly as shown below:

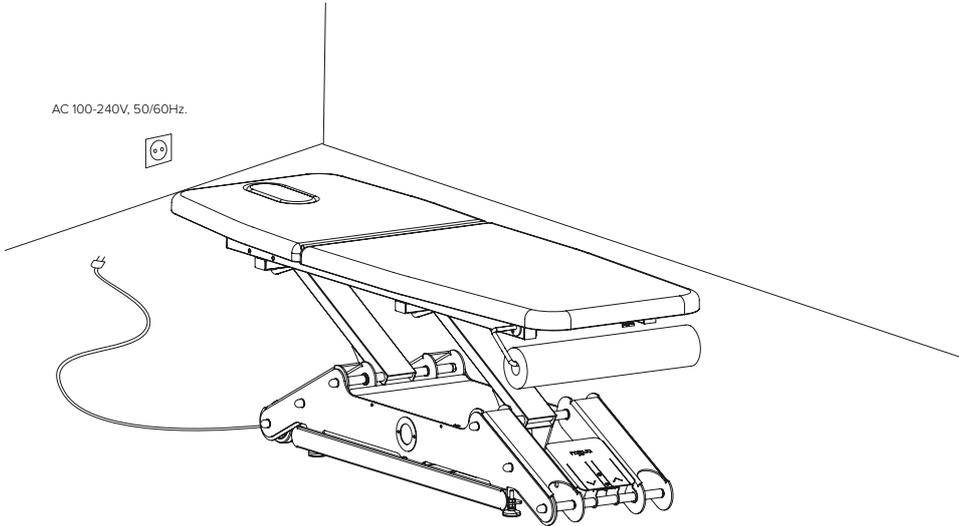


Specifications

Product size	196(+/- 20 cm with movement) x 68 x 47 / 87 cm
Package size	206 x 72 x 52 cm
Highest position	87 cm
Lowest position	47 cm
Net weight	107 / 136 Kg
Gross weight	141 Kg
Power	AC 100 – 120V, 50/60 Hz · AC 220 – 240V, 50/60 Hz
Max power	300W
Environment temperature	0 °C - +40 °C
Relative humidity	30% - 90%
Max weight supported	200 Kg




Product dimensions


N'GO301

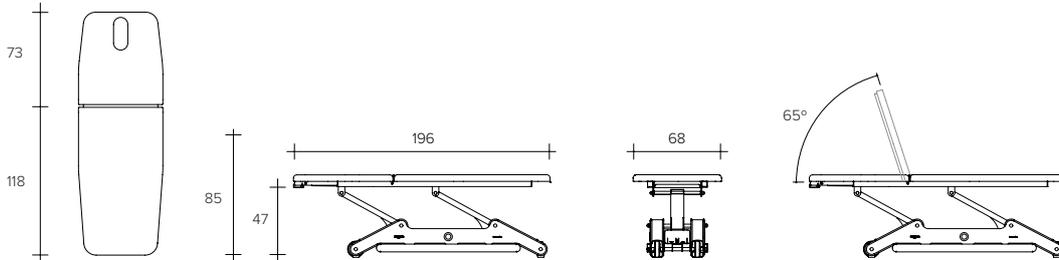


N'GO303

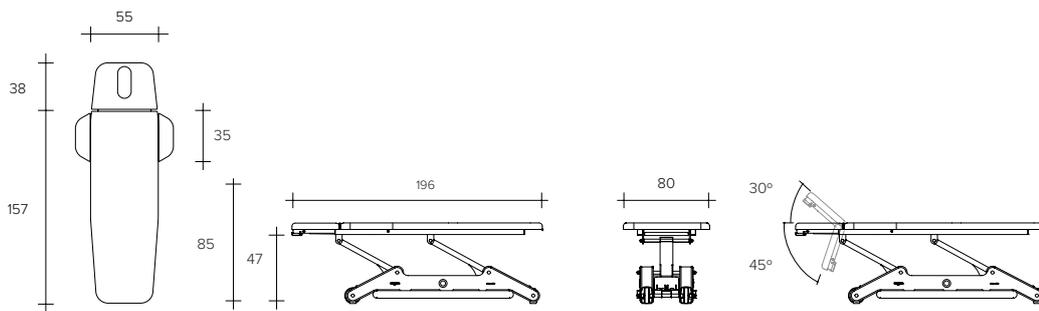

N'GO304F / 304

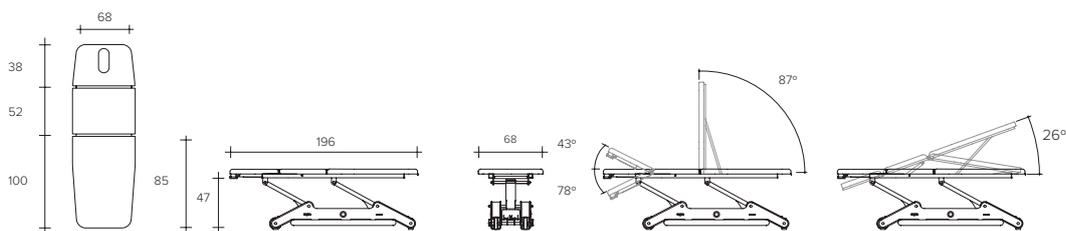

N'GO308

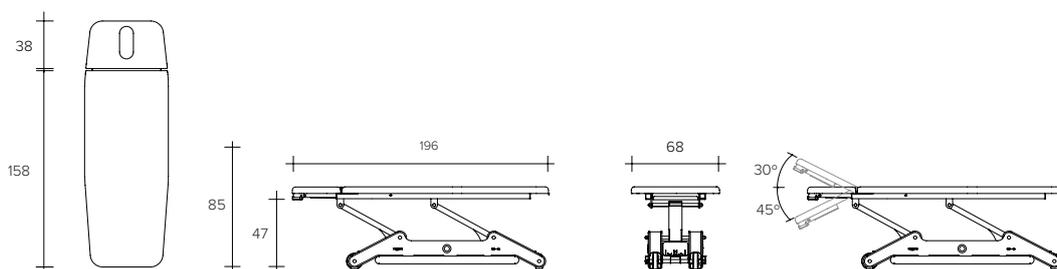



Essential Settings

 Optional

Section motors


If your table is equipped with more than one motor to adjust one of the sections, is possible to adjust them by the push button box placed at both sides of the table. If a handcontroller is included, the motors can also be adjusted by the buttons on the handcontroller.




N'RUN304 button panel for controlling middle section engine.

Circuit diagram

N'GO301 / 303 / 304F / 308

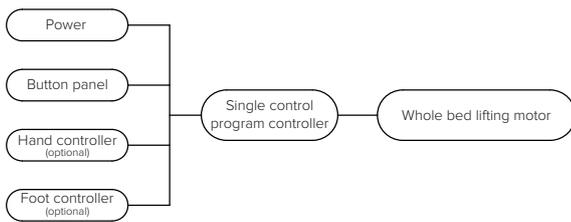

N'GO304

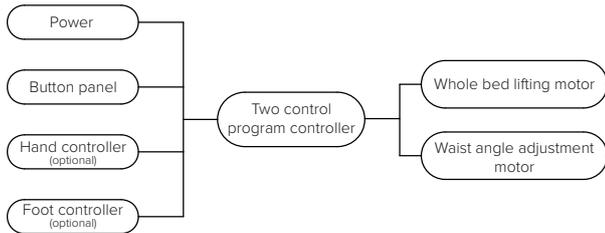




Essential Settings

Safety Lock 9

The table can be locked or unlocked using the key on the base. If the key is in ON position, the table is unlocked and ready to use. If the position is OFF, the table is locked and the controls don't work.

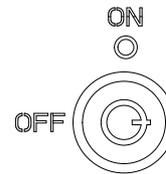

Maintenance

Cleaning

Clean the table frame using a damp white cloth, clear water and a domestic free-of-alcohol household cleaner. To clean the vinylic upholstery, mix clear water with a mild soap, soak in water and remove excess of water. Next, rinse with clear water and dry. Clean only in one direction.

Important: Lacquers, strong cleaners, xylene-based solutions, acetone or MEK cause immediate damage and contribute to the deterioration of the material. The use of such cleaners is at the owner's risk.

Maintenance

Naggura tables have been designed for an intensive use during long time thanks to its stability, reliability and durability. However, we advise to keep the table in perfect conditions and make a correct use of it. Unplug the table if you will not use it for a long period of time.

Environment information

If this symbol appears on electrical products or on its packaging, this means that they must be recycled properly. Please ensure that you are well informed of the local rules and regulations regarding to the removal of equipment and accessories.



Troubleshooting

The top sections cannot be adjusted

The supporting gas springs might "stick" if the couch has not been used for a long time


- In such a case, push the corresponding lever with your hand, wait for a few seconds and pull the table section upwards with the other hand.

The electric adjustments do not work properly

- Check the power cable for any damage.
- Check all the plugs and connections.

Persisting problems

If problems persist, please contact the dealer and provide the serial number and model of the table as they appear on the backrest gaslift.

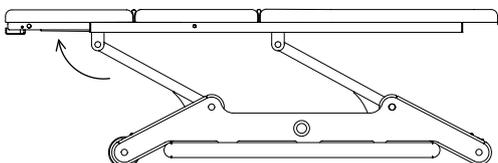



Our products are subject to continuous technical improvements. Naggura reserves the right to change or modify the product and contents without prior notice.

Naggura

Pol. Ind. La Valldan
Camí de Garreta, 92
08600 Berga
Barcelona - Spain
T. +34 93 824 90 20
E. info@naggura.com